

# LIFEPAK® 500 AED

## Addendum to Operating Instructions

---

### CAUTION!

- **Possible Equipment Damage.** This device may be damaged by mechanical or physical abuse such as immersion in water or dropping the device. If the device has been abused, remove it from use and contact a qualified service technician.
- **Possible Therapy Connector Damage.** The LIFEPAK 500 AED has an eight (8) year expected life. Using this device beyond 8 years may result in excessive wear and damage of components within the Therapy Connector. Devices should not be in use beyond the eight (8) year expected life.

### Troubleshooting During Patient Care

Observation	Possible Cause	Corrective Action
<b>CONNECT ELECTRODES</b> message appears.	Inadequate connection to AED.	<ul style="list-style-type: none"> <li>• Check for complete insertion of connector to AED.</li> <li>• Remove and reinstall electrode connector to AED.</li> </ul>
	Electrode does not adhere properly to the patient.	<ul style="list-style-type: none"> <li>• Press electrodes firmly on patient's skin.</li> <li>• Clean, shave, and dry the patient's skin as recommended.</li> </ul>
	Electrodes are dry, damaged, or out-of-date.	<ul style="list-style-type: none"> <li>• Replace the electrodes.</li> </ul>

#### Stryker

Emergency Care  
 11811 Willows Road NE  
 P.O. Box 97006  
 Redmond, WA 98073-9706  
 Tel 425 867 4000  
 Fax 425 867 4121  
[strykeremergencycare.com](http://strykeremergencycare.com)