


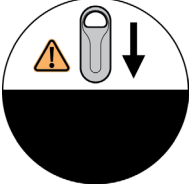




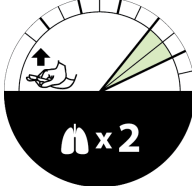


TrueCPR™

COACHING DEVICE / QUICK REFERENCE CARD

COACHING DEVICE / QUICK REFERENCE CARD

COACHING DEVICE / QUICK REFERENCE CARD

 <p>Power button Power button Power button</p>	 <p>Reposition back pad Reposition back pad Reposition back pad</p>
 <p>Mute button Mute button Mute button</p>	
 <p>Airway button Airway button Airway button</p>	
 <p>Low battery indicator Low battery indicator Low battery indicator</p>	
 <p>Countdown to ventilations Countdown to ventilations Countdown to ventilations</p>	 <p>Ventilation prompt Ventilation prompt Ventilation prompt</p>

If the device cannot be positioned correctly on the patient or fails to operate, remove the device and perform CPR unaided.

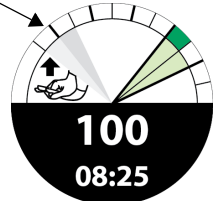
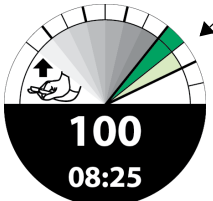
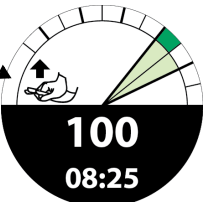




If the device cannot be positioned correctly on the patient or fails to operate, remove the device and perform CPR unaided.

If the device cannot be positioned correctly on the patient or fails to operate, remove the device and perform CPR unaided.

See the *Instructions for Use* for complete operating instructions.

See the *Instructions for Use* for complete operating instructions.

See the *Instructions for Use* for complete operating instructions.

 <p>100 08:25</p> <p>Compression in process Compression in process Compression in process</p>	 <p>100 08:25</p> <p>Sufficient depth Sufficient depth Sufficient depth</p>
 <p>100 08:25</p> <p>Recoil reminder Recoil reminder Recoil reminder</p>	<p>IMPORTANT: Be sure to release each compression completely. Device does not indicate complete release.</p> <p>IMPORTANT: Be sure to release each compression completely. Device does not indicate complete release.</p> <p>IMPORTANT: Be sure to release each compression completely. Device does not indicate complete release.</p>
 <p>100 08:25</p> <p>Compression too shallow Compression too shallow Compression too shallow</p>	 <p>100 08:25</p> <p>Compression > 6 cm Compression > 6 cm Compression > 6 cm</p>
 <p>✓ ↓ 93%</p> <p>W 104</p> <p>Review 1: Press Power + Airway Review 1: Press Power + Airway Review 1: Press Power + Airway</p>	 <p>🕒 16:59</p> <p>🕒 ↓ 91%</p> <p>Review 2: Press Mute Review 2: Press Mute Review 2: Press Mute</p>

Physio-Control, Inc.
11811 Willows Rd. NE
Redmond, WA, 98052 USA

©2013 Physio-Control, Inc.

USA Rx Only

PN 3311753-002