

LIFEPAK® CR2 Defibrillator

Supplemental Operating Instructions

Refer to the *LIFEPAK CR2 Defibrillator Operating Instructions* for complete directions for use.

Warning

Premature Battery Depletion. If the lid or lid magnet is missing, the LIFEPAK CR2 battery can become prematurely depleted.

Troubleshooting Tips

This table explains problem conditions that you may encounter while using the defibrillator.

OBSERVATION	POSSIBLE CAUSE	CORRECTIVE ACTION
Defibrillator does not provide voice prompts or beeping tones after you open the lid (turn it on).	Lid magnet is missing	<ul style="list-style-type: none"> Use the ON/OFF button to turn the defibrillator on if required for emergency use. Contact qualified service personnel.
	Depleted battery	<ul style="list-style-type: none"> Replace the battery immediately. If a replacement is not available, order a new battery immediately.
	Speaker system failure	<ul style="list-style-type: none"> Contact qualified service personnel.
Voice prompts continue after lid is closed.	Lid magnet is missing	<ul style="list-style-type: none"> Use the ON/OFF button to turn the defibrillator off. Contact qualified service personnel.

Stryker

Emergency Care
 11811 Willows Road NE
 Redmond, WA 98052 USA
 Tel: 1 800 STRYKER
 Toll Free (USA only): 1 800 732 3081
 Fax: 425 867 4121
strykeremergencycare.com

